

Бикарбонат

NTB аналог NAD+

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN **BiCarb** Test Code: IL_ **BCarbd** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Type: **Serum**

Reaction Cycle: ☒ Standard ☐ Extended

Reporting Unit: ☒ ммоль /л * ☐ User Define Decimal Points: Conv. Factor:

Methodology: Type: ☒ End Point ☐ Rate Measuring Point: - ☒ Photometric: ☐ 1 Wavelength ☒ 2 Wavelength Primary: Secondary:

Correction Constant: Slope: Intercept:

Test Name: NN **BiCarb** Test Code: IL_ **BCarbd** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Volume:

	Sample	Dilution	
	Volume	Sample Vol.	Diluent Vol.
1	2.0	0.0	0.0
2	2.0	0.0	0.0
3	3.0	0.0	0.0
4	0.0	0.0	0.0

 Diluent: Diluent Warning Limit: Tests

Reagent Volume: R1: **BiCarb-1** R2: **BiCarb-2**

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	200	0.0	<input checked="" type="checkbox"/>	20	No Control
R2			<input checked="" type="checkbox"/>	20	No Control

Sampling Condition:

	Condition No.	1	2	3	4
<input type="checkbox"/>	First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Below N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Above N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	User Range L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	User Range H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Noise	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Prozone	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	HIGH!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	ABS!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

Ranges		Limits																																																							
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>22</td> <td>29</td> </tr> <tr> <td>Female</td> <td>22</td> <td>29</td> </tr> <tr> <td>Other</td> <td>22</td> <td>29</td> </tr> </tbody> </table>		Normal Range				Lower	Upper	Male	22	29	Female	22	29	Other	22	29	<table border="1"> <thead> <tr> <th>Reaction Slope</th> <th>Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Negative <input checked="" type="radio"/> Positive </td> <td> <input checked="" type="radio"/> Above <input type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>0</td> <td>%</td> </tr> </thead></table> </td> <td> <table border="1"> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> <td> <table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table> </td> </tr> </thead></table></td></tr></tbody> </table>		Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>0</td> <td>%</td> </tr> </thead></table>	Non-Linear Limit		0	%	<table border="1"> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </table>	2500.0	mAbs	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> <td> <table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table> </td> </tr> </thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table>		Judge Point		0	
Normal Range																																																									
	Lower	Upper																																																							
Male	22	29																																																							
Female	22	29																																																							
Other	22	29																																																							
Reaction Slope	Absorbance Limit																																																								
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																																								
<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>0</td> <td>%</td> </tr> </thead></table>	Non-Linear Limit		0	%	<table border="1"> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </table>	2500.0	mAbs																																																		
Non-Linear Limit																																																									
0	%																																																								
2500.0	mAbs																																																								
<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> <td> <table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table> </td> </tr> </thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table>		Judge Point		0																																
Serum Index Limits																																																									
	Limit																																																								
Hemolysis	0.0																																																								
Icterus	0.0																																																								
Lipemia	0.0																																																								
Prozone Limit																																																									
<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table>	Limit	Equation	0.0	none																																																				
Limit	Equation																																																								
0.0	none																																																								
<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table>		Judge Point		0																																																					
Judge Point																																																									
0																																																									
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>4</td> <td>50</td> </tr> </tbody> </table>		User Range		Lower	Upper	4	50																																																		
User Range																																																									
Lower	Upper																																																								
4	50																																																								
<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>4</td> <td>50</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	4	50																																																		
ValidRange																																																									
Lower	Upper																																																								
4	50																																																								
<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <td> <input type="radio"/> On <input checked="" type="radio"/> Off </td> <td></td> </tr> </thead></table>		Qualitative		<input type="radio"/> On <input checked="" type="radio"/> Off																																																					
Qualitative																																																									
<input type="radio"/> On <input checked="" type="radio"/> Off																																																									

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																																		
NN	BiCarb	IL	BCarbd																																									
Calibration				Condition				Calibration/R-Blank Limit																																				
<table border="1"> <thead> <tr> <th colspan="2">Method</th> <th colspan="2">Curve Type</th> </tr> </thead> <tbody> <tr> <td>1-point</td> <td></td> <td>Linear</td> <td></td> </tr> </tbody> </table>				Method		Curve Type		1-point		Linear		<table border="1"> <thead> <tr> <th colspan="2">Repeats</th> <th colspan="2">Stability</th> </tr> </thead> <tbody> <tr> <td>2</td> <td></td> <td></td> <td>days</td> </tr> </tbody> </table>				Repeats		Stability		2			days	<table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>				No																
Method		Curve Type																																										
1-point		Linear																																										
Repeats		Stability																																										
2			days																																									
No																																												
<table border="1"> <thead> <tr> <th colspan="3">Calibrator</th> </tr> <tr> <th></th> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>StdBiCar-1</td> <td>**</td> </tr> <tr> <td></td> <td></td> <td></td> </tr> <tr> <td></td> <td></td> <td></td> </tr> <tr> <td>4</td> <td></td> <td></td> </tr> <tr> <td>5</td> <td></td> <td></td> </tr> </tbody> </table>				Calibrator				Calibrator	Conc	1	StdBiCar-1	**							4			5			<table border="1"> <thead> <tr> <th colspan="2">Condition</th> </tr> </thead> <tbody> <tr> <td><input checked="" type="checkbox"/></td> <td>Reagent Blank</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Auto Reagent Blank by Bottle</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Ask for calibration when reagent lot changes</td> </tr> </tbody> </table>				Condition		<input checked="" type="checkbox"/>	Reagent Blank	<input type="checkbox"/>	Auto Reagent Blank by Bottle	<input type="checkbox"/>	Ask for calibration when reagent lot changes	<table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> </thead> <tbody> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </tbody> </table>				R-Blank Limit		2500.0	mAbs
Calibrator																																												
	Calibrator	Conc																																										
1	StdBiCar-1	**																																										
4																																												
5																																												
Condition																																												
<input checked="" type="checkbox"/>	Reagent Blank																																											
<input type="checkbox"/>	Auto Reagent Blank by Bottle																																											
<input type="checkbox"/>	Ask for calibration when reagent lot changes																																											
R-Blank Limit																																												
2500.0	mAbs																																											
				<table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> </thead> <tbody> <tr> <td>20.0</td> <td>%</td> </tr> </tbody> </table>				Cal Reps Range		20.0	%																																	
Cal Reps Range																																												
20.0	%																																											
				<table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> </thead> <tbody> <tr> <td>100.0</td> <td>mAbs</td> </tr> </tbody> </table>				Min Cal Reps		100.0	mAbs																																	
Min Cal Reps																																												
100.0	mAbs																																											
				<table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> </thead> <tbody> <tr> <td>20</td> <td>%</td> </tr> </tbody> </table>				Factor Change		20	%																																	
Factor Change																																												
20	%																																											
				<table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>%</td> </tr> </tbody> </table>				M-Point Curve Fit		0.0	%																																	
M-Point Curve Fit																																												
0.0	%																																											

Бикарбонат

NTB аналог NAD+



* -Рекомендуемый параметр.

** - Вводится из паспорта к стандарту.

Тип калибровки: Линейная, по стандарту "Стандартный раствор бикарбоната"
(Bicarbonate Standard FS).

Контроль по TruLab Bicarbonate.