

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN **GluHK** Test Code: IL_ **GluHK-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Type
Serum

Reaction Cycle
Standard Extended

Reporting Unit
mmol/L * Decimal Points: 2
User Define
Conv. Factor: 0.000

Methodology
Type: End Point Rate
Measuring Point: 16 - 33
Photometric: 1 Wavelength 2 Wavelength
Primary: 340 Secondary: 546
Correction Constant: Slope: 1.000 Intercept: 0.000

Test Name: NN **GluHK** Test Code: IL_ **GluHK-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Volume

	Sample	Dilution	
	Volume	Sample Vol.	Diluent Vol.
1	2.0	0.0	0.0
2	2.0	20.0	80.0
3	4.0	0.0	0.0
4	0.0	0.0	0.0

Diluent:
Diluent Warning Limit: 0 Tests

Reagent Volume

R1: GluHKd1 R2: GluHKd2

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	200	0.0	✓	20	No Control
R2	50	0.0	✓	20	No Control

Sampling Condition

	Condition No.	1	2	3	4
First Run		✓			
Samp.Vol.Reduction			✓		
Below N-Range					
Above N-Range					
Panic L					
Panic H			✓		
User Range L					
User Range H	✓		✓		
Noise					
Prozone					
HIGH!	✓		✓		
ABS!	✓		✓		

Ranges		Limits																																																																																	
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>3.9</td> <td>6.4</td> </tr> <tr> <td>Female</td> <td>3.9</td> <td>6.4</td> </tr> <tr> <td>Other</td> <td>3.9</td> <td>6.4</td> </tr> </tbody> </table>		Normal Range				Lower	Upper	Male	3.9	6.4	Female	3.9	6.4	Other	3.9	6.4	<table border="1"> <thead> <tr> <th>Reaction Slope</th> <th>Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Negative <input checked="" type="radio"/> Positive </td> <td> <input checked="" type="radio"/> Above <input type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>0</td> <td>%</td> </tr> </thead></table> </td> <td> <table border="1"> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead> <tbody> <tr> <td> <table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table> </td> <td> <table border="1"> <tr> <td>Judge Point</td> </tr> <tr> <td>0</td> </tr> </table> </td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.1</td> <td>28</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.1</td> <td>28</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </thead> </table> </td> <td colspan="2"></td> </tr> </tbody> </table>		Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>0</td> <td>%</td> </tr> </thead></table>	Non-Linear Limit		0	%	<table border="1"> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </table>	2500.0	mAbs	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead> <tbody> <tr> <td> <table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table> </td> <td> <table border="1"> <tr> <td>Judge Point</td> </tr> <tr> <td>0</td> </tr> </table> </td> </tr> </tbody> </table>	Prozone Limit		<input type="radio"/> Above	<input checked="" type="radio"/> Below	<table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table>	Limit	Equation	0.0	none	<table border="1"> <tr> <td>Judge Point</td> </tr> <tr> <td>0</td> </tr> </table>	Judge Point	0	<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.1</td> <td>28</td> </tr> </tbody> </table>		User Range		Lower	Upper	0.1	28			<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.1</td> <td>28</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	0.1	28			<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </thead> </table>		Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off		
Normal Range																																																																																			
	Lower	Upper																																																																																	
Male	3.9	6.4																																																																																	
Female	3.9	6.4																																																																																	
Other	3.9	6.4																																																																																	
Reaction Slope	Absorbance Limit																																																																																		
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																																																																		
<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>0</td> <td>%</td> </tr> </thead></table>	Non-Linear Limit		0	%	<table border="1"> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </table>	2500.0	mAbs																																																																												
Non-Linear Limit																																																																																			
0	%																																																																																		
2500.0	mAbs																																																																																		
<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead> <tbody> <tr> <td> <table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table> </td> <td> <table border="1"> <tr> <td>Judge Point</td> </tr> <tr> <td>0</td> </tr> </table> </td> </tr> </tbody> </table>	Prozone Limit		<input type="radio"/> Above	<input checked="" type="radio"/> Below	<table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table>	Limit	Equation	0.0	none	<table border="1"> <tr> <td>Judge Point</td> </tr> <tr> <td>0</td> </tr> </table>	Judge Point	0																																																											
Serum Index Limits																																																																																			
	Limit																																																																																		
Hemolysis	0.0																																																																																		
Icterus	0.0																																																																																		
Lipemia	0.0																																																																																		
Prozone Limit																																																																																			
<input type="radio"/> Above	<input checked="" type="radio"/> Below																																																																																		
<table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table>	Limit	Equation	0.0	none	<table border="1"> <tr> <td>Judge Point</td> </tr> <tr> <td>0</td> </tr> </table>	Judge Point	0																																																																												
Limit	Equation																																																																																		
0.0	none																																																																																		
Judge Point																																																																																			
0																																																																																			
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.1</td> <td>28</td> </tr> </tbody> </table>		User Range		Lower	Upper	0.1	28																																																																												
User Range																																																																																			
Lower	Upper																																																																																		
0.1	28																																																																																		
<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.1</td> <td>28</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	0.1	28																																																																												
ValidRange																																																																																			
Lower	Upper																																																																																		
0.1	28																																																																																		
<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </thead> </table>		Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																														
Qualitative																																																																																			
<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																																		

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																																		
NN	GluHK	IL_	GluHK-d																																									
Calibration				Condition				Calibration/R-Blank Limit																																				
<table border="1"> <thead> <tr> <th colspan="2">Method</th> <th colspan="2">Curve Type</th> </tr> </thead> <tbody> <tr> <td>1-point</td> <td></td> <td>Linear</td> <td></td> </tr> </tbody> </table>				Method		Curve Type		1-point		Linear		<table border="1"> <thead> <tr> <th colspan="2">Repeats</th> <th colspan="2">Stability</th> </tr> </thead> <tbody> <tr> <td>2</td> <td></td> <td></td> <td>days</td> </tr> </tbody> </table>				Repeats		Stability		2			days	<table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>				No																
Method		Curve Type																																										
1-point		Linear																																										
Repeats		Stability																																										
2			days																																									
No																																												
<table border="1"> <thead> <tr> <th colspan="3">Calibrator</th> </tr> <tr> <th></th> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>TruCalU</td> <td>**</td> </tr> <tr> <td>2</td> <td></td> <td></td> </tr> <tr> <td>3</td> <td></td> <td></td> </tr> <tr> <td>4</td> <td></td> <td></td> </tr> <tr> <td>5</td> <td></td> <td></td> </tr> </tbody> </table>				Calibrator				Calibrator	Conc	1	TruCalU	**	2			3			4			5			<table border="1"> <thead> <tr> <th colspan="2">Condition</th> </tr> </thead> <tbody> <tr> <td><input checked="" type="checkbox"/></td> <td>Reagent Blank</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Auto Reagent Blank by Bottle</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Ask for calibration when reagent lot changes</td> </tr> </tbody> </table>				Condition		<input checked="" type="checkbox"/>	Reagent Blank	<input type="checkbox"/>	Auto Reagent Blank by Bottle	<input type="checkbox"/>	Ask for calibration when reagent lot changes	<table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> </thead> <tbody> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </tbody> </table>				R-Blank Limit		2500.0	mAbs
Calibrator																																												
	Calibrator	Conc																																										
1	TruCalU	**																																										
2																																												
3																																												
4																																												
5																																												
Condition																																												
<input checked="" type="checkbox"/>	Reagent Blank																																											
<input type="checkbox"/>	Auto Reagent Blank by Bottle																																											
<input type="checkbox"/>	Ask for calibration when reagent lot changes																																											
R-Blank Limit																																												
2500.0	mAbs																																											
				<table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> </thead> <tbody> <tr> <td>20.0</td> <td>%</td> </tr> </tbody> </table>				Cal Reps Range		20.0	%																																	
Cal Reps Range																																												
20.0	%																																											
				<table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> </thead> <tbody> <tr> <td>50.0</td> <td>mAbs</td> </tr> </tbody> </table>				Min Cal Reps		50.0	mAbs																																	
Min Cal Reps																																												
50.0	mAbs																																											
				<table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> </thead> <tbody> <tr> <td>20</td> <td>%</td> </tr> </tbody> </table>				Factor Change		20	%																																	
Factor Change																																												
20	%																																											
				<table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>%</td> </tr> </tbody> </table>				M-Point Curve Fit		0.0	%																																	
M-Point Curve Fit																																												
0.0	%																																											

Глюкоза

НК (гексокиназный метод)



* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Тип калибровки: Линейная (калибратор TruCalU) . Контроль по TruLab N и TruLab P.