

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN **K** Test Code: IL_ **K-d** Measure Sample Reagent Ranges Limits Calibration

Sample Type
Serum

Reaction Cycle
☒ Standard ☐ Extended

Reporting Unit
☒ ммоль/л * ☐ User Define
Decimal Points: 2
Conv. Factor: 0.000

Methodology
Type
☐ End Point ☒ Rate
Measuring Point
26 - ☒ 33
Photometric
☐ 1 Wavelength ☒ 2 Wavelength
Primary: 340 **Secondary**: 546
Correction Constant
Slope: 1.000 Intercept: 0.000

Test Name: NN **K** Test Code: IL_ **K-d** Measure Sample Reagent Ranges Limits Calibration

Sample Volume

	Sample Volume	Dilution Sample Vol.	Dilution Diluent Vol.
1	3.0	0.0	0.0
2	2.0	0.0	0.0
3	4.0	0.0	0.0
4			

Diluent:
Diluent Warning Limit: 0 Tests

Reagent Volume
R1: **K-d1** R2: **K-d2**

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	120	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	40	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition

Condition No.	1	2	3	4
First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
Below N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Above N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Panic L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Panic H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
User Range L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/> User Range H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
Noise	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Prozone	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/> HIGH!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/> ABS!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

Ranges		Limits																																																							
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>3.5</td> <td>5.4</td> </tr> <tr> <td>Female</td> <td>3.5</td> <td>5.4</td> </tr> <tr> <td>Other</td> <td>3.5</td> <td>5.4</td> </tr> </tbody> </table>		Normal Range				Lower	Upper	Male	3.5	5.4	Female	3.5	5.4	Other	3.5	5.4	<table border="1"> <thead> <tr> <th>Reaction Slope</th> <th>Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td> <input checked="" type="radio"/> Negative <input type="radio"/> Positive </td> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>25</td> <td>%</td> </tr> </thead></table> </td> <td> <table border="1"> <tr> <td>700.0</td> <td>mAbs</td> </tr> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> <td> <table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table> </td> </tr> </thead></table></td></tr></tbody> </table>		Reaction Slope	Absorbance Limit	<input checked="" type="radio"/> Negative <input type="radio"/> Positive	<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>25</td> <td>%</td> </tr> </thead></table>	Non-Linear Limit		25	%	<table border="1"> <tr> <td>700.0</td> <td>mAbs</td> </tr> </table>	700.0	mAbs	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> <td> <table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table> </td> </tr> </thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table>		Judge Point		0	
Normal Range																																																									
	Lower	Upper																																																							
Male	3.5	5.4																																																							
Female	3.5	5.4																																																							
Other	3.5	5.4																																																							
Reaction Slope	Absorbance Limit																																																								
<input checked="" type="radio"/> Negative <input type="radio"/> Positive	<input type="radio"/> Above <input checked="" type="radio"/> Below																																																								
<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>25</td> <td>%</td> </tr> </thead></table>	Non-Linear Limit		25	%	<table border="1"> <tr> <td>700.0</td> <td>mAbs</td> </tr> </table>	700.0	mAbs																																																		
Non-Linear Limit																																																									
25	%																																																								
700.0	mAbs																																																								
<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> <td> <table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table> </td> </tr> </thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table>		Judge Point		0																																
Serum Index Limits																																																									
	Limit																																																								
Hemolysis	0.0																																																								
Icterus	0.0																																																								
Lipemia	0.0																																																								
Prozone Limit																																																									
<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table>	Limit	Equation	0.0	none																																																				
Limit	Equation																																																								
0.0	none																																																								
<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table>		Judge Point		0																																																					
Judge Point																																																									
0																																																									
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>10</td> </tr> </tbody> </table>		User Range		Lower	Upper	2	10																																																		
User Range																																																									
Lower	Upper																																																								
2	10																																																								
<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>10</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	2	10																																																		
ValidRange																																																									
Lower	Upper																																																								
2	10																																																								
<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <td> <input type="radio"/> On <input checked="" type="radio"/> Off </td> <td></td> </tr> </thead> </table>		Qualitative		<input type="radio"/> On <input checked="" type="radio"/> Off																																																					
Qualitative																																																									
<input type="radio"/> On <input checked="" type="radio"/> Off																																																									

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																																																															
NN	K	IL	K-d																																																																						
Calibration				Condition				Calibration/R-Blank Limit																																																																	
<table border="1"> <thead> <tr> <th colspan="2">Method</th> <th colspan="2">Curve Type</th> </tr> </thead> <tbody> <tr> <td>M-point</td> <td></td> <td>Linear</td> <td></td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Repeats</th> </tr> <tr> <td>2</td> <td></td> </tr> </thead></table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Stability</th> </tr> <tr> <td></td> <td>days</td> </tr> </thead></table> </td> </tr> <tr> <td colspan="4"> <table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> <tr> <td></td> <td></td> </tr> </thead></table> </td> <td colspan="4"> <table border="1"> <thead> <tr> <th colspan="2">Calibrator</th> </tr> <tr> <th></th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>Std-K-1 **</td> </tr> <tr> <td>2</td> <td>Std-K-2 **</td> </tr> <tr> <td>3</td> <td></td> </tr> <tr> <td>4</td> <td></td> </tr> <tr> <td>5</td> <td></td> </tr> </tbody> </table> </td> </tr> </tbody> </table>				Method		Curve Type		M-point		Linear		<table border="1"> <thead> <tr> <th colspan="2">Repeats</th> </tr> <tr> <td>2</td> <td></td> </tr> </thead></table>		Repeats		2		<table border="1"> <thead> <tr> <th colspan="2">Stability</th> </tr> <tr> <td></td> <td>days</td> </tr> </thead></table>		Stability			days	<table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> <tr> <td></td> <td></td> </tr> </thead></table>				No				<table border="1"> <thead> <tr> <th colspan="2">Calibrator</th> </tr> <tr> <th></th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>Std-K-1 **</td> </tr> <tr> <td>2</td> <td>Std-K-2 **</td> </tr> <tr> <td>3</td> <td></td> </tr> <tr> <td>4</td> <td></td> </tr> <tr> <td>5</td> <td></td> </tr> </tbody> </table>				Calibrator			Conc	1	Std-K-1 **	2	Std-K-2 **	3		4		5		<table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </thead></table>				R-Blank Limit		2500.0	mAbs	<table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> <tr> <td>20.0</td> <td>%</td> </tr> </thead></table>				Cal Reps Range		20.0	%	<table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> <tr> <td>100.0</td> <td>mAbs</td> </tr> </thead></table>				Min Cal Reps		100.0	mAbs
Method		Curve Type																																																																							
M-point		Linear																																																																							
<table border="1"> <thead> <tr> <th colspan="2">Repeats</th> </tr> <tr> <td>2</td> <td></td> </tr> </thead></table>		Repeats		2		<table border="1"> <thead> <tr> <th colspan="2">Stability</th> </tr> <tr> <td></td> <td>days</td> </tr> </thead></table>		Stability			days																																																														
Repeats																																																																									
2																																																																									
Stability																																																																									
	days																																																																								
<table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> <tr> <td></td> <td></td> </tr> </thead></table>				No				<table border="1"> <thead> <tr> <th colspan="2">Calibrator</th> </tr> <tr> <th></th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>Std-K-1 **</td> </tr> <tr> <td>2</td> <td>Std-K-2 **</td> </tr> <tr> <td>3</td> <td></td> </tr> <tr> <td>4</td> <td></td> </tr> <tr> <td>5</td> <td></td> </tr> </tbody> </table>				Calibrator			Conc	1	Std-K-1 **	2	Std-K-2 **	3		4		5																																																	
No																																																																									
Calibrator																																																																									
	Conc																																																																								
1	Std-K-1 **																																																																								
2	Std-K-2 **																																																																								
3																																																																									
4																																																																									
5																																																																									
R-Blank Limit																																																																									
2500.0	mAbs																																																																								
Cal Reps Range																																																																									
20.0	%																																																																								
Min Cal Reps																																																																									
100.0	mAbs																																																																								
<table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <td>20</td> <td>%</td> </tr> </thead></table>				Factor Change		20	%	<table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <td>0.0</td> <td>%</td> </tr> </thead></table>				M-Point Curve Fit		0.0	%																																																										
Factor Change																																																																									
20	%																																																																								
M-Point Curve Fit																																																																									
0.0	%																																																																								

Калий

Ферментативный



* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Тип калибровки: Линейная, по двум стандартам из набора.

Контроль по TruLab N и TruLab P.