

Свободные жирные кислоты (NEFA)

ферментативный

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN NEFA Test Code: IL_ NEFA-d

Buttons: Measure, Sample Reagent, Ranges Limits, Calibration

Sample Type: Serum

Reaction Cycle: Standard, Extended

Reporting Unit: ммоль /л * Decimal Points: 1

Methodology: Type: End Point, Rate; Measuring Point: 16 - 33; Photometric: 1 Wavelength, 2 Wavelength; Primary: 546, Secondary: 700; Correction Constant: Slope: 1.000, Intercept: 0.000

Test Name: NN NEFA Test Code: IL_ NEFA-d

Buttons: Measure, Sample Reagent, Ranges Limits, Calibration

Sample Volume: Sample, Dilution, Diluent, Diluent Warning Limit

Reagent Volume: R1, R2

Sampling Condition: Condition No., 1, 2, 3, 4

Condition No.	1	2	3	4
First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
Below N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Above N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Panic L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Panic H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
User Range L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
User Range H	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
Noise	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Prozone	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
HIGH!	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
ABS!	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

Свободные жирные кислоты (NEFA)

ферментативный

Ranges		Limits																																		
<table border="1"><thead><tr><th colspan="3">Normal Range</th></tr><tr><th></th><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>Male</td><td>0.1</td><td>0.6</td></tr><tr><td>Female</td><td>0.1</td><td>0.45</td></tr><tr><td>Other</td><td>0.1</td><td>0.6</td></tr></tbody></table>		Normal Range				Lower	Upper	Male	0.1	0.6	Female	0.1	0.45	Other	0.1	0.6	<table border="1"><thead><tr><th>Reaction Slope</th><th>Absorbance Limit</th></tr></thead><tbody><tr><td><input type="radio"/> Negative <input checked="" type="radio"/> Positive</td><td><input checked="" type="radio"/> Above <input type="radio"/> Below</td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table></td><td><table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td>Limit: 0.0</td><td>Equation: none</td></tr><tr><td colspan="2">Judge Point: 0</td></tr></thead></table></td></tr></tbody></table>		Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table>	Non-Linear Limit		0	%	<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td>Limit: 0.0</td><td>Equation: none</td></tr><tr><td colspan="2">Judge Point: 0</td></tr></thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		Limit: 0.0	Equation: none	Judge Point: 0	
Normal Range																																				
	Lower	Upper																																		
Male	0.1	0.6																																		
Female	0.1	0.45																																		
Other	0.1	0.6																																		
Reaction Slope	Absorbance Limit																																			
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																			
<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table>	Non-Linear Limit		0	%	<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td>Limit: 0.0</td><td>Equation: none</td></tr><tr><td colspan="2">Judge Point: 0</td></tr></thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		Limit: 0.0	Equation: none	Judge Point: 0																								
Non-Linear Limit																																				
0	%																																			
Prozone Limit																																				
<input type="radio"/> Above <input checked="" type="radio"/> Below																																				
Limit: 0.0	Equation: none																																			
Judge Point: 0																																				

	User Range			------------	-------		Lower	Upper		0	3.0				
	ValidRange			------------	-------		Lower	Upper		0	3.0				
Qualitative															
☐ On ☒ Off															

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																																		
NN	NEFA	IL	NEFA-d																																									
Calibration				Condition				Calibration/R-Blank Limit																																				
<table border="1"><thead><tr><th colspan="2">Method</th></tr><tr><td>1-point</td><td></td></tr></thead><tbody></tbody></table>				Method		1-point		<table border="1"><thead><tr><th colspan="2">Curve Type</th></tr><tr><td>Linear</td><td></td></tr></thead><tbody></tbody></table>				Curve Type		Linear		<table border="1"><thead><tr><th colspan="2">R-Blank Limit</th></tr><tr><td>2500.0</td><td>mAbs</td></tr></thead><tbody></tbody></table>				R-Blank Limit		2500.0	mAbs																					
Method																																												
1-point																																												
Curve Type																																												
Linear																																												
R-Blank Limit																																												
2500.0	mAbs																																											
<table border="1"><thead><tr><th colspan="2">Repeats</th></tr><tr><td>2</td><td></td></tr></thead><tbody></tbody></table>				Repeats		2		<table border="1"><thead><tr><th colspan="2">Stability</th></tr><tr><td></td><td>days</td></tr></thead><tbody></tbody></table>				Stability			days	<table border="1"><thead><tr><th colspan="2">Cal Reps Range</th></tr><tr><td>20.0</td><td>%</td></tr></thead><tbody></tbody></table>				Cal Reps Range		20.0	%																					
Repeats																																												
2																																												
Stability																																												
	days																																											
Cal Reps Range																																												
20.0	%																																											
<table border="1"><thead><tr><th colspan="2">No</th></tr><tr><td></td><td></td></tr></thead><tbody></tbody></table>				No				<table border="1"><thead><tr><th colspan="2">Min Cal Reps</th></tr><tr><td>150.0</td><td>mAbs</td></tr></thead><tbody></tbody></table>				Min Cal Reps		150.0	mAbs	<table border="1"><thead><tr><th colspan="2">Factor Change</th></tr><tr><td>20</td><td>%</td></tr></thead><tbody></tbody></table>				Factor Change		20	%																					
No																																												
Min Cal Reps																																												
150.0	mAbs																																											
Factor Change																																												
20	%																																											
<table border="1"><thead><tr><th colspan="3">Calibrator</th></tr><tr><th></th><th>Calibrator</th><th>Conc</th></tr></thead><tbody><tr><td>1</td><td>StdNEFA</td><td>**</td></tr><tr><td>2</td><td></td><td></td></tr><tr><td>3</td><td></td><td></td></tr><tr><td>4</td><td></td><td></td></tr><tr><td>5</td><td></td><td></td></tr></tbody></table>				Calibrator				Calibrator	Conc	1	StdNEFA	**	2			3			4			5			<table border="1"><thead><tr><th colspan="2">M-Point Curve Fit</th></tr><tr><td>0.0</td><td>%</td></tr></thead><tbody></tbody></table>				M-Point Curve Fit		0.0	%	<table border="1"><thead><tr><th colspan="2">Condition</th></tr><tr><td><input checked="" type="checkbox"/> Reagent Blank</td><td></td></tr><tr><td><input type="checkbox"/> Auto Reagent Blank by Bottle</td><td></td></tr><tr><td><input type="checkbox"/> Ask for calibration when reagent lot changes</td><td></td></tr></thead><tbody></tbody></table>				Condition		<input checked="" type="checkbox"/> Reagent Blank		<input type="checkbox"/> Auto Reagent Blank by Bottle		<input type="checkbox"/> Ask for calibration when reagent lot changes	
Calibrator																																												
	Calibrator	Conc																																										
1	StdNEFA	**																																										
2																																												
3																																												
4																																												
5																																												
M-Point Curve Fit																																												
0.0	%																																											
Condition																																												
<input checked="" type="checkbox"/> Reagent Blank																																												
<input type="checkbox"/> Auto Reagent Blank by Bottle																																												
<input type="checkbox"/> Ask for calibration when reagent lot changes																																												

Свободные жирные кислоты (NEFA)

ферментативный



* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Тип калибровки: Линейная, по стандарту, "Стандарт свободных жирных кислот"
(NEFA Standard FS(Non-esterified fatty acids) .

Контроль по TruLab L (липидный) уровень 1 и 2.