

Параметры для ввода в программу анализатора iLab-Taurus

Test Name
NN **Urea**

Test Code
IL_ **Urea-d**

Measure

Sample Reagent

Ranges Limits

Calibration

Sample Type
Serum

Reaction Cycle
☒ Standard
 ☐ Extended

Reporting Unit
☒ ммоль/л *

Decimal Points
 2

☐ User Define

Conv.Factor
 0.000

Methodology

Type
☐ End Point
 ☒ Rate

Measuring Point

22

 - ☒

29

Photometric
☐ 1 Wavelength
 ☒ 2 Wavelength

Primary
340

Secondary
546

Correction Constant

Slope
1.000

Intercept
0.000

Test Name
NN **Urea**

Test Code
IL_ **Urea-d**

Measure

Sample Reagent

Ranges Limits

Calibration

Sample Volume

	Sample	Dilution	
	Volume	Sample Vol.	Diluent Vol.
1	2.0	0.0	0.0
2	2.0	10.0	90.0
3	4.0	0.0	0.0
4	2.0	4.0	96.0

Diluent

Diluent Warning Limit
0 Tests

Reagent Volume

R1
Urea-d1

R2
Urea-d2

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	200	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	50	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition

	Condition No.	1	2	3	4
<input type="checkbox"/>	First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Below N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Above N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	User Range L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	User Range H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Noise	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Prozone	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	HIGH!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	ABS!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

Мочевина

Urease-GLDH UV (уреазный-глутаматдегидрогеназный метод)



Ranges		Limits																																		
<table border="1"><thead><tr><th colspan="3">Normal Range</th></tr><tr><th></th><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>Male</td><td>3.2</td><td>7.3</td></tr><tr><td>Female</td><td>2.6</td><td>6.7</td></tr><tr><td>Other</td><td>2.6</td><td>7.3</td></tr></tbody></table>		Normal Range				Lower	Upper	Male	3.2	7.3	Female	2.6	6.7	Other	2.6	7.3	<table border="1"><thead><tr><th>Reaction Slope</th><th>Absorbance Limit</th></tr></thead><tbody><tr><td><input checked="" type="radio"/> Negative <input type="radio"/> Positive</td><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>30</td><td>%</td></tr></thead><tbody></tbody></table></td><td><table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td>Limit: 0.0</td><td>Equation: none</td></tr><tr><td colspan="2">Judge Point: 0</td></tr></thead></table></td></tr></tbody></table>		Reaction Slope	Absorbance Limit	<input checked="" type="radio"/> Negative <input type="radio"/> Positive	<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>30</td><td>%</td></tr></thead><tbody></tbody></table>	Non-Linear Limit		30	%	<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td>Limit: 0.0</td><td>Equation: none</td></tr><tr><td colspan="2">Judge Point: 0</td></tr></thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		Limit: 0.0	Equation: none	Judge Point: 0	
Normal Range																																				
	Lower	Upper																																		
Male	3.2	7.3																																		
Female	2.6	6.7																																		
Other	2.6	7.3																																		
Reaction Slope	Absorbance Limit																																			
<input checked="" type="radio"/> Negative <input type="radio"/> Positive	<input type="radio"/> Above <input checked="" type="radio"/> Below																																			
<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>30</td><td>%</td></tr></thead><tbody></tbody></table>	Non-Linear Limit		30	%	<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td>Limit: 0.0</td><td>Equation: none</td></tr><tr><td colspan="2">Judge Point: 0</td></tr></thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		Limit: 0.0	Equation: none	Judge Point: 0																								
Non-Linear Limit																																				
30	%																																			
Prozone Limit																																				
<input type="radio"/> Above <input checked="" type="radio"/> Below																																				
Limit: 0.0	Equation: none																																			
Judge Point: 0																																				

	User Range			------------	-------		Lower	Upper		0.3	50				
	ValidRange			------------	-------		Lower	Upper		0.3	50				
Qualitative															
☐ On ☒ Off															

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																																																																							
NN	Urea	IL	Urea-d																																																																														
Calibration		Condition		Calibration/R-Blank Limit																																																																													
<table border="1"><thead><tr><th colspan="2">Method</th></tr><tr><td>1-point</td><td></td></tr></thead><tbody></tbody></table>		Method		1-point		<table border="1"><thead><tr><th colspan="2">Curve Type</th></tr><tr><td>Linear</td><td></td></tr></thead><tbody></tbody></table>		Curve Type		Linear		<table border="1"><thead><tr><th colspan="2">Repeats</th></tr><tr><td>2</td><td></td></tr></thead><tbody></tbody></table>		Repeats		2		<table border="1"><thead><tr><th colspan="2">Stability</th></tr><tr><td></td><td>days</td></tr></thead><tbody></tbody></table>		Stability			days	<table border="1"><thead><tr><th colspan="2">No</th></tr><tr><td></td><td></td></tr></thead><tbody></tbody></table>		No				<table border="1"><thead><tr><th colspan="2">Calibrator</th></tr><tr><th></th><th>Calibrator</th><th>Conc</th></tr></thead><tbody><tr><td>1</td><td>TruCalU</td><td>**</td></tr><tr><td>2</td><td></td><td></td></tr><tr><td>3</td><td></td><td></td></tr><tr><td>4</td><td></td><td></td></tr><tr><td>5</td><td></td><td></td></tr></tbody></table>		Calibrator			Calibrator	Conc	1	TruCalU	**	2			3			4			5			<table border="1"><thead><tr><th colspan="2">R-Blank Limit</th></tr><tr><td>2500.0</td><td>mAbs</td></tr></thead><tbody></tbody></table>		R-Blank Limit		2500.0	mAbs	<table border="1"><thead><tr><th colspan="2">Cal Reps Range</th></tr><tr><td>20.0</td><td>%</td></tr></thead><tbody></tbody></table>		Cal Reps Range		20.0	%	<table border="1"><thead><tr><th colspan="2">Min Cal Reps</th></tr><tr><td>0</td><td>mAbs</td></tr></thead><tbody></tbody></table>		Min Cal Reps		0	mAbs	<table border="1"><thead><tr><th colspan="2">Factor Change</th></tr><tr><td>20</td><td>%</td></tr></thead><tbody></tbody></table>		Factor Change		20	%	<table border="1"><thead><tr><th colspan="2">M-Point Curve Fit</th></tr><tr><td>0.0</td><td>%</td></tr></thead><tbody></tbody></table>		M-Point Curve Fit		0.0	%
Method																																																																																	
1-point																																																																																	
Curve Type																																																																																	
Linear																																																																																	
Repeats																																																																																	
2																																																																																	
Stability																																																																																	
	days																																																																																
No																																																																																	
Calibrator																																																																																	
	Calibrator	Conc																																																																															
1	TruCalU	**																																																																															
2																																																																																	
3																																																																																	
4																																																																																	
5																																																																																	
R-Blank Limit																																																																																	
2500.0	mAbs																																																																																
Cal Reps Range																																																																																	
20.0	%																																																																																
Min Cal Reps																																																																																	
0	mAbs																																																																																
Factor Change																																																																																	
20	%																																																																																
M-Point Curve Fit																																																																																	
0.0	%																																																																																
<input checked="" type="checkbox"/> Reagent Blank		<input type="checkbox"/> Auto Reagent Blank by Bottle		<input type="checkbox"/> Ask for calibration when reagent lot changes																																																																													

Мочевина

Urease-GLDH UV (уреазный-глутаматдегидрогеназный метод)



* -Рекомендуемый параметр.

****** - Вводится из паспорта калибратору.

Тип калибровки: Линейная (калибратор TruCalU).

Контроль по TruLab N и TruLab P.