

р-Амилаза (панкреатическая)

IFCC, Zasz, EPS-G7 (метод с использованием мальтогентазида (EPS-G7))



Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN **pAmy** Test Code: IL_ **pAmy-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Type
Serum

Reaction Cycle
☒ Standard ☐ Extended

Reporting Unit
☒ Ед/л * ☐ User Define
 Decimal Points: 1
 Conv.Factor: 0.000

Methodology
Type
☐ End Point ☒ Rate
Measuring Point
 22 - ☒ 31
Photometric
☐ 1 Wavelength ☒ 2 Wavelength
Primary: 405 **Secondary**: 546
Correction Constant
 Slope: 1.000 Intercept: 0.000

Test Name: NN **pAmy** Test Code: IL_ **pAmy-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Volume

	Sample	Dilution	
	Volume	Sample Vol.	Diluent Vol.
1	3.0	0.0	0.0
2	3.0	10.0	90.0
3	6.0	0.0	0.0
4	2.0	5.0	95.0

Diluent:
Diluent Warning Limit: 0 Tests

Reagent Volume

R1: **pAmy-d1** R2: **pAmy-d2**

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	140	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	35	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition

	Condition No.	1	2	3	4
<input type="checkbox"/>	First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Below N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Above N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	User Range L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	User Range H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Noise	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Prozone	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	HIGH!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	ABS!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

р-Амилаза (панкреатическая)

IFCC, Zasz, EPS-G7 (метод с использованием мальтогентазида (EPS-G7))



Ranges			Limits																																																																																									
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>0</td> <td>53</td> </tr> <tr> <td>Female</td> <td>0</td> <td>53</td> </tr> <tr> <td>Other</td> <td>0</td> <td>53</td> </tr> </tbody> </table>			Normal Range				Lower	Upper	Male	0	53	Female	0	53	Other	0	53	<table border="1"> <thead> <tr> <th>Reaction Slope</th> <th>Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Negative <input checked="" type="radio"/> Positive </td> <td> <input checked="" type="radio"/> Above <input type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>25</td> <td></td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>2500.0</td> <td></td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>3</td> <td>2000</td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th>Index</th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>3</td> <td>2000</td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Equation</th> </tr> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <th>On</th> <th>Off</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/></td> <td><input checked="" type="radio"/></td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>0</td> <td></td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>25</td> <td></td> </tr> </tbody> </table>	Non-Linear Limit		Value	%	25		<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>2500.0</td> <td></td> </tr> </tbody> </table>	Prozone Limit		Value	%	2500.0		<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>3</td> <td>2000</td> </tr> </tbody> </table>			User Range		Lower	Upper	3	2000	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th>Index</th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits		Index	Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>3</td> <td>2000</td> </tr> </tbody> </table>			ValidRange		Lower	Upper	3	2000	<table border="1"> <thead> <tr> <th colspan="2">Equation</th> </tr> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>		Equation		Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <th>On</th> <th>Off</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/></td> <td><input checked="" type="radio"/></td> </tr> </tbody> </table>			Qualitative		On	Off	<input type="radio"/>	<input checked="" type="radio"/>	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>0</td> <td></td> </tr> </tbody> </table>		Judge Point		Value	%	0	
Normal Range																																																																																												
	Lower	Upper																																																																																										
Male	0	53																																																																																										
Female	0	53																																																																																										
Other	0	53																																																																																										
Reaction Slope	Absorbance Limit																																																																																											
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																																																																											
<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>25</td> <td></td> </tr> </tbody> </table>	Non-Linear Limit		Value	%	25		<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>2500.0</td> <td></td> </tr> </tbody> </table>	Prozone Limit		Value	%	2500.0																																																																																
Non-Linear Limit																																																																																												
Value	%																																																																																											
25																																																																																												
Prozone Limit																																																																																												
Value	%																																																																																											
2500.0																																																																																												
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>3</td> <td>2000</td> </tr> </tbody> </table>			User Range		Lower	Upper	3	2000	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th>Index</th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits		Index	Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0																																																																								
User Range																																																																																												
Lower	Upper																																																																																											
3	2000																																																																																											
Serum Index Limits																																																																																												
Index	Limit																																																																																											
Hemolysis	0.0																																																																																											
Icterus	0.0																																																																																											
Lipemia	0.0																																																																																											
<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>3</td> <td>2000</td> </tr> </tbody> </table>			ValidRange		Lower	Upper	3	2000	<table border="1"> <thead> <tr> <th colspan="2">Equation</th> </tr> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>		Equation		Limit	Equation	0.0	none																																																																												
ValidRange																																																																																												
Lower	Upper																																																																																											
3	2000																																																																																											
Equation																																																																																												
Limit	Equation																																																																																											
0.0	none																																																																																											
<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <th>On</th> <th>Off</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/></td> <td><input checked="" type="radio"/></td> </tr> </tbody> </table>			Qualitative		On	Off	<input type="radio"/>	<input checked="" type="radio"/>	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>0</td> <td></td> </tr> </tbody> </table>		Judge Point		Value	%	0																																																																													
Qualitative																																																																																												
On	Off																																																																																											
<input type="radio"/>	<input checked="" type="radio"/>																																																																																											
Judge Point																																																																																												
Value	%																																																																																											
0																																																																																												

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																																						
NN	pAmy	IL	pAmy-d																																													
Calibration				Condition				Calibration/R-Blank Limit																																								
<table border="1"> <thead> <tr> <th colspan="2">Method</th> <th colspan="2">Curve Type</th> </tr> <tr> <th>Value</th> <th>Unit</th> <th>Value</th> <th>Unit</th> </tr> </thead> <tbody> <tr> <td>1-point</td> <td></td> <td>Linear</td> <td></td> </tr> </tbody> </table>				Method		Curve Type		Value	Unit	Value	Unit	1-point		Linear		<table border="1"> <thead> <tr> <th colspan="2">Repeats</th> <th colspan="2">Stability</th> </tr> <tr> <th>Value</th> <th>Unit</th> <th>Value</th> <th>Unit</th> </tr> </thead> <tbody> <tr> <td>2</td> <td></td> <td></td> <td>days</td> </tr> </tbody> </table>				Repeats		Stability		Value	Unit	Value	Unit	2			days	<table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> <tr> <th>Value</th> <th>Unit</th> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>				No		Value	Unit									
Method		Curve Type																																														
Value	Unit	Value	Unit																																													
1-point		Linear																																														
Repeats		Stability																																														
Value	Unit	Value	Unit																																													
2			days																																													
No																																																
Value	Unit																																															
<table border="1"> <thead> <tr> <th colspan="3">Calibrator</th> </tr> <tr> <th></th> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>TruCalU</td> <td>**</td> </tr> <tr> <td>2</td> <td></td> <td></td> </tr> <tr> <td>3</td> <td></td> <td></td> </tr> <tr> <td>4</td> <td></td> <td></td> </tr> <tr> <td>5</td> <td></td> <td></td> </tr> </tbody> </table>				Calibrator				Calibrator	Conc	1	TruCalU	**	2			3			4			5			<table border="1"> <thead> <tr> <th colspan="2">Condition</th> </tr> <tr> <th>Value</th> <th>Unit</th> </tr> </thead> <tbody> <tr> <td><input checked="" type="checkbox"/></td> <td>Reagent Blank</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Auto Reagent Blank by Bottle</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Ask for calibration when reagent lot changes</td> </tr> </tbody> </table>				Condition		Value	Unit	<input checked="" type="checkbox"/>	Reagent Blank	<input type="checkbox"/>	Auto Reagent Blank by Bottle	<input type="checkbox"/>	Ask for calibration when reagent lot changes	<table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> <tr> <th>Value</th> <th>Unit</th> </tr> </thead> <tbody> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </tbody> </table>				R-Blank Limit		Value	Unit	2500.0	mAbs
Calibrator																																																
	Calibrator	Conc																																														
1	TruCalU	**																																														
2																																																
3																																																
4																																																
5																																																
Condition																																																
Value	Unit																																															
<input checked="" type="checkbox"/>	Reagent Blank																																															
<input type="checkbox"/>	Auto Reagent Blank by Bottle																																															
<input type="checkbox"/>	Ask for calibration when reagent lot changes																																															
R-Blank Limit																																																
Value	Unit																																															
2500.0	mAbs																																															
				<table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>20.0</td> <td></td> </tr> </tbody> </table>				Cal Reps Range		Value	%	20.0																																				
Cal Reps Range																																																
Value	%																																															
20.0																																																
				<table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> <tr> <th>Value</th> <th>Unit</th> </tr> </thead> <tbody> <tr> <td>50.0</td> <td>mAbs</td> </tr> </tbody> </table>				Min Cal Reps		Value	Unit	50.0	mAbs																																			
Min Cal Reps																																																
Value	Unit																																															
50.0	mAbs																																															
				<table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>20</td> <td></td> </tr> </tbody> </table>				Factor Change		Value	%	20																																				
Factor Change																																																
Value	%																																															
20																																																
				<table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td></td> </tr> </tbody> </table>				M-Point Curve Fit		Value	%	0.0																																				
M-Point Curve Fit																																																
Value	%																																															
0.0																																																

р-Амилаза (панкреатическая)

IFCC, Zasz, EPS-G7 (метод с использованием мальтогентазида (EPS-G7))



* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Тип калибровки: Линейная (калибратор TruCalU) или по фактору (см инструкцию).

Контроль по TruLab N и TruLab P.