

# Имуноглобулин М, Ig-M

ИтипоTD (Иммунотурбидиметрический)

## Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN IgM Test Code: IL\_ IgM-d [Measure] [Sample Reagent] [Ranges Limits] [Calibration]

Sample Type: Serum

Reaction Cycle: ☒ Standard ☐ Extended

Reporting Unit: ☒ г/л \* ☐ User Define   
Decimal Points: 2   
Conv.Factor: 0.000

Methodology:   
Type: ☒ End Point ☐ Rate   
Measuring Point: 16 - ☒ 29   
Photometric: ☐ 1 Wavelength ☒ 2 Wavelength   
Primary: 410 Secondary: 750   
Correction Constant: Slope: 1.000 Intercept: 0.000

Test Name: NN IgM Test Code: IL\_ IgM-d [Measure] [Sample Reagent] [Ranges Limits] [Calibration]

Sample Volume: 

	Sample	Dilution	
	Volume	Sample Vol.	Diluent Vol.
1	2.0	0.0	0.0
2	2.0	30.0	60.0
3	4.0	0.0	0.0
4	0.0	0.0	0.0

  
Diluent:   
Diluent Warning Limit: 0 Tests

Reagent Volume:   
R1: IgM-1 R2: IgM-2   

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	250	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	50	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition: 

	Condition No.	1	2	3	4
<input type="checkbox"/>	First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Below N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Above N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic H	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	User Range L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	User Range H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Noise	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Prozone	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	HIGH!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	ABS!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

# Имуноглобулин М, Ig-M

ИммуноТD (Иммунотурбидиметрический)

Ranges		Limits																																												
<table border="1"><thead><tr><th colspan="3">Normal Range</th></tr><tr><th></th><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>Male</td><td>0.4</td><td>2.3</td></tr><tr><td>Female</td><td>0.4</td><td>2.3</td></tr><tr><td>Other</td><td>0.4</td><td>2.3</td></tr></tbody></table>		Normal Range				Lower	Upper	Male	0.4	2.3	Female	0.4	2.3	Other	0.4	2.3	<table border="1"><thead><tr><th>Reaction Slope</th><th>Absorbance Limit</th></tr></thead><tbody><tr><td><input type="radio"/> Negative <input checked="" type="radio"/> Positive</td><td><input checked="" type="radio"/> Above <input type="radio"/> Below</td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>100</td><td>%</td></tr></thead></table></td><td><table border="1"><thead><tr><td>2600.0</td><td>mAbs</td></tr></thead></table></td></tr><tr><td colspan="2"><table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td><td></td></tr><tr><td colspan="2"><table border="1"><thead><tr><th>Judge Point</th></tr><tr><td>0</td></tr></thead></table></td></tr></thead></table></td></tr></tbody></table>		Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>100</td><td>%</td></tr></thead></table>	Non-Linear Limit		100	%	<table border="1"><thead><tr><td>2600.0</td><td>mAbs</td></tr></thead></table>	2600.0	mAbs	<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td><td></td></tr><tr><td colspan="2"><table border="1"><thead><tr><th>Judge Point</th></tr><tr><td>0</td></tr></thead></table></td></tr></thead></table>		Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none		<table border="1"><thead><tr><th>Judge Point</th></tr><tr><td>0</td></tr></thead></table>		Judge Point	0
Normal Range																																														
	Lower	Upper																																												
Male	0.4	2.3																																												
Female	0.4	2.3																																												
Other	0.4	2.3																																												
Reaction Slope	Absorbance Limit																																													
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																													
<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>100</td><td>%</td></tr></thead></table>	Non-Linear Limit		100	%	<table border="1"><thead><tr><td>2600.0</td><td>mAbs</td></tr></thead></table>	2600.0	mAbs																																							
Non-Linear Limit																																														
100	%																																													
2600.0	mAbs																																													
<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td><td></td></tr><tr><td colspan="2"><table border="1"><thead><tr><th>Judge Point</th></tr><tr><td>0</td></tr></thead></table></td></tr></thead></table>		Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none		<table border="1"><thead><tr><th>Judge Point</th></tr><tr><td>0</td></tr></thead></table>		Judge Point	0																															
Prozone Limit																																														
<input type="radio"/> Above <input checked="" type="radio"/> Below																																														
<table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none																																										
Limit	Equation																																													
0.0	none																																													
<table border="1"><thead><tr><th>Judge Point</th></tr><tr><td>0</td></tr></thead></table>		Judge Point	0																																											
Judge Point																																														
0																																														

User Range	
Lower	Upper
0.25	8.00

ValidRange	
Lower	Upper
0.25	8.00

Qualitative	
<input type="radio"/> On <input checked="" type="radio"/> Off	

Serum Index Limits	
	Limit
Hemolysis	0.0
Icterus	0.0
Lipemia	0.0

Test Name		Test Code		Measure	Sample Reagent	Ranges Limits	Calibration
NN	IgM	IL	IgM-d				

Calibration	Condition	Calibration/R-Blank Limit																																																											
<table border="1"><thead><tr><th>Method</th><th>Curve Type</th></tr></thead><tbody><tr><td>M-point</td><td>5P Log</td></tr></tbody></table> <table border="1"><thead><tr><th>Repeats</th><th>No</th></tr></thead><tbody><tr><td>2</td><td>6</td></tr></tbody></table> <table border="1"><thead><tr><th colspan="2">Stability</th></tr><tr><td></td><td>days</td></tr></thead></table> <table border="1"><thead><tr><th colspan="3">Calibrator</th></tr><tr><th></th><th>Calibrator</th><th>Conc</th></tr></thead><tbody><tr><td>1</td><td>NaCl*</td><td>0.0</td></tr><tr><td>2</td><td>TC-prot-1</td><td>**</td></tr><tr><td>3</td><td>TC-prot-2</td><td>**</td></tr><tr><td>4</td><td>TC-prot-3</td><td>**</td></tr><tr><td>5</td><td>TC-prot-4</td><td>**</td></tr><tr><td>6</td><td>TC-prot-5</td><td>**</td></tr></tbody></table>	Method	Curve Type	M-point	5P Log	Repeats	No	2	6	Stability			days	Calibrator				Calibrator	Conc	1	NaCl*	0.0	2	TC-prot-1	**	3	TC-prot-2	**	4	TC-prot-3	**	5	TC-prot-4	**	6	TC-prot-5	**	<table border="1"><tbody><tr><td><input type="checkbox"/> Reagent Blank</td></tr><tr><td><input type="checkbox"/> Auto Reagent Blank by Bottle</td></tr><tr><td><input type="checkbox"/> Ask for calibration when reagent lot changes</td></tr></tbody></table>	<input type="checkbox"/> Reagent Blank	<input type="checkbox"/> Auto Reagent Blank by Bottle	<input type="checkbox"/> Ask for calibration when reagent lot changes	<table border="1"><thead><tr><th colspan="2">R-Blank Limit</th></tr><tr><td>2500.0</td><td>mAbs</td></tr></thead><table border="1"><thead><tr><th colspan="2">Cal Reps Range</th></tr><tr><td>15.0</td><td>%</td></tr></thead><table border="1"><thead><tr><th colspan="2">Min Cal Reps</th></tr><tr><td>200.0</td><td>mAbs</td></tr></thead><table border="1"><thead><tr><th colspan="2">Factor Change</th></tr><tr><td>35</td><td>%</td></tr></thead><table border="1"><thead><tr><th colspan="2">M-Point Curve Fit</th></tr><tr><td>0.0</td><td>%</td></tr></thead></table></table></table></table></table>	R-Blank Limit		2500.0	mAbs	Cal Reps Range		15.0	%	Min Cal Reps		200.0	mAbs	Factor Change		35	%	M-Point Curve Fit		0.0	%
Method	Curve Type																																																												
M-point	5P Log																																																												
Repeats	No																																																												
2	6																																																												
Stability																																																													
	days																																																												
Calibrator																																																													
	Calibrator	Conc																																																											
1	NaCl*	0.0																																																											
2	TC-prot-1	**																																																											
3	TC-prot-2	**																																																											
4	TC-prot-3	**																																																											
5	TC-prot-4	**																																																											
6	TC-prot-5	**																																																											
<input type="checkbox"/> Reagent Blank																																																													
<input type="checkbox"/> Auto Reagent Blank by Bottle																																																													
<input type="checkbox"/> Ask for calibration when reagent lot changes																																																													
R-Blank Limit																																																													
2500.0	mAbs																																																												
Cal Reps Range																																																													
15.0	%																																																												
Min Cal Reps																																																													
200.0	mAbs																																																												
Factor Change																																																													
35	%																																																												
M-Point Curve Fit																																																													
0.0	%																																																												

# Имуноглобулин М, Ig-M

ИттипoTD (Иммунотурбидиметрический)



\* -Рекомендуемый параметр.

\*\* - Вводится из паспорта калибратору.

Калибровка многоточечная (6 точек, 0+5 уровней) Spline или 5P Log,  
(калибратор TruCal Protein).

Контроль TruLab Protein уровень 1 и 2.