

## Параметры для ввода в программу анализатора Beckman-Coulter AU 480

Parameters		Specific Test Parameters																																																																																																																					
General	LIH	ISE	HbA1c	Calculated Test	Range																																																																																																																		
Test Name: <b>Fe</b> < > Type: <b>Serum</b> Operation <b>Yes</b>																																																																																																																							
<table border="0"> <tr> <td>Sample Volume</td> <td><b>9</b> <math>\mu</math>L</td> <td>Dilution</td> <td><b>0</b> <math>\mu</math>L</td> <td>OD Limit</td> <td></td> </tr> <tr> <td>Pre-Dilution Rate</td> <td><b>1</b> <math>\nabla</math></td> <td></td> <td></td> <td>Min.OD</td> <td><b>-2.0</b> Max.OD <b>2.5</b></td> </tr> <tr> <td>Rgt. Volume</td> <td>R1(R1-1) <b>90</b> <math>\mu</math>L</td> <td>Dilution</td> <td><b>0</b> <math>\mu</math>L</td> <td>Reagent OD Limit</td> <td></td> </tr> <tr> <td></td> <td></td> <td></td> <td></td> <td>First Low</td> <td><b>-2.0</b> High <b>2.5</b></td> </tr> <tr> <td></td> <td></td> <td></td> <td></td> <td>Last Low</td> <td><b>-2.0</b> High <b>2.5</b></td> </tr> <tr> <td></td> <td>R2(R2-1) <b>23</b> <math>\mu</math>L</td> <td>Dilution</td> <td><b>0</b> <math>\mu</math>L</td> <td>Dynamic Range Low</td> <td><b>0</b> High <b>179</b></td> </tr> <tr> <td></td> <td></td> <td></td> <td></td> <td>Correlation Factor A</td> <td><b>1</b> B <b>0</b></td> </tr> <tr> <td></td> <td></td> <td></td> <td></td> <td>Factor for Maker A</td> <td><b>1</b> B <b>0</b></td> </tr> <tr> <td>Wavelength</td> <td>Pri <b>600</b> <math>\nabla</math>nm</td> <td>Sec.</td> <td><b>700</b> <math>\nabla</math>nm</td> <td>Onboard Stability Period</td> <td><b>999</b> Day <input type="text"/> Hour</td> </tr> <tr> <td>Method</td> <td><b>END</b> <math>\nabla</math></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Reaction Slope</td> <td><b>+</b> <math>\nabla</math></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Measuring Point1 First</td> <td><b>0</b></td> <td>Last</td> <td><b>27</b></td> <td></td> <td></td> </tr> <tr> <td>Measuring Point2 First</td> <td><b>0</b></td> <td>Last</td> <td><b>10</b></td> <td></td> <td></td> </tr> <tr> <td>Linearity Limit</td> <td><input type="text"/> %</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Lag Time Check</td> <td><input type="text"/> <math>\nabla</math></td> <td></td> <td></td> <td></td> <td></td> </tr> </table>						Sample Volume	<b>9</b> $\mu$ L	Dilution	<b>0</b> $\mu$ L	OD Limit		Pre-Dilution Rate	<b>1</b> $\nabla$			Min.OD	<b>-2.0</b> Max.OD <b>2.5</b>	Rgt. Volume	R1(R1-1) <b>90</b> $\mu$ L	Dilution	<b>0</b> $\mu$ L	Reagent OD Limit						First Low	<b>-2.0</b> High <b>2.5</b>					Last Low	<b>-2.0</b> High <b>2.5</b>		R2(R2-1) <b>23</b> $\mu$ L	Dilution	<b>0</b> $\mu$ L	Dynamic Range Low	<b>0</b> High <b>179</b>					Correlation Factor A	<b>1</b> B <b>0</b>					Factor for Maker A	<b>1</b> B <b>0</b>	Wavelength	Pri <b>600</b> $\nabla$ nm	Sec.	<b>700</b> $\nabla$ nm	Onboard Stability Period	<b>999</b> Day <input type="text"/> Hour	Method	<b>END</b> $\nabla$					Reaction Slope	<b>+</b> $\nabla$					Measuring Point1 First	<b>0</b>	Last	<b>27</b>			Measuring Point2 First	<b>0</b>	Last	<b>10</b>			Linearity Limit	<input type="text"/> %					Lag Time Check	<input type="text"/> $\nabla$																												
Sample Volume	<b>9</b> $\mu$ L	Dilution	<b>0</b> $\mu$ L	OD Limit																																																																																																																			
Pre-Dilution Rate	<b>1</b> $\nabla$			Min.OD	<b>-2.0</b> Max.OD <b>2.5</b>																																																																																																																		
Rgt. Volume	R1(R1-1) <b>90</b> $\mu$ L	Dilution	<b>0</b> $\mu$ L	Reagent OD Limit																																																																																																																			
				First Low	<b>-2.0</b> High <b>2.5</b>																																																																																																																		
				Last Low	<b>-2.0</b> High <b>2.5</b>																																																																																																																		
	R2(R2-1) <b>23</b> $\mu$ L	Dilution	<b>0</b> $\mu$ L	Dynamic Range Low	<b>0</b> High <b>179</b>																																																																																																																		
				Correlation Factor A	<b>1</b> B <b>0</b>																																																																																																																		
				Factor for Maker A	<b>1</b> B <b>0</b>																																																																																																																		
Wavelength	Pri <b>600</b> $\nabla$ nm	Sec.	<b>700</b> $\nabla$ nm	Onboard Stability Period	<b>999</b> Day <input type="text"/> Hour																																																																																																																		
Method	<b>END</b> $\nabla$																																																																																																																						
Reaction Slope	<b>+</b> $\nabla$																																																																																																																						
Measuring Point1 First	<b>0</b>	Last	<b>27</b>																																																																																																																				
Measuring Point2 First	<b>0</b>	Last	<b>10</b>																																																																																																																				
Linearity Limit	<input type="text"/> %																																																																																																																						
Lag Time Check	<input type="text"/> $\nabla$																																																																																																																						
<table border="0"> <tr> <td colspan="2">Parameters</td> <td colspan="4">Specific Test Parameters</td> </tr> <tr> <td>General</td> <td>LIH</td> <td>ISE</td> <td>HbA1c</td> <td>Calculated Test</td> <td>Range</td> </tr> </table>						Parameters		Specific Test Parameters				General	LIH	ISE	HbA1c	Calculated Test	Range																																																																																																						
Parameters		Specific Test Parameters																																																																																																																					
General	LIH	ISE	HbA1c	Calculated Test	Range																																																																																																																		
Test Name: <b>Fe</b> < > Type: <b>Serum</b>																																																																																																																							
<table border="0"> <tr> <td>Value/Flag:</td> <td><input type="text"/> <math>\nabla</math></td> <td>Level L:</td> <td><input type="text"/></td> <td>Level H:</td> <td><input type="text"/></td> </tr> <tr> <td colspan="6">Specific Ranges:</td> </tr> <tr> <td></td> <td>Sex</td> <td>Year</td> <td>Month</td> <td>Year</td> <td>Month</td> </tr> <tr> <td><input type="checkbox"/> 1.</td> <td><input type="text"/> <math>\nabla</math></td> <td><input type="text"/></td> <td><input type="text"/></td> <td><input type="text"/></td> <td><input type="text"/></td> </tr> <tr> <td><input type="checkbox"/> 2.</td> <td><input type="text"/> <math>\nabla</math></td> <td><input type="text"/></td> <td><input type="text"/></td> <td><input type="text"/></td> <td><input type="text"/></td> </tr> <tr> <td><input type="checkbox"/> 3.</td> <td><input type="text"/> <math>\nabla</math></td> <td><input type="text"/></td> <td><input type="text"/></td> <td><input type="text"/></td> <td><input type="text"/></td> </tr> <tr> <td><input type="checkbox"/> 4.</td> <td><input type="text"/> <math>\nabla</math></td> <td><input type="text"/></td> <td><input type="text"/></td> <td><input type="text"/></td> <td><input type="text"/></td> </tr> <tr> <td><input type="checkbox"/> 5.</td> <td><input type="text"/> <math>\nabla</math></td> <td><input type="text"/></td> <td><input type="text"/></td> <td><input type="text"/></td> <td><input type="text"/></td> </tr> <tr> <td><input type="checkbox"/> 6.</td> <td><input type="text"/> <math>\nabla</math></td> <td><input type="text"/></td> <td><input type="text"/></td> <td><input type="text"/></td> <td><input type="text"/></td> </tr> <tr> <td><input type="checkbox"/> 7.</td> <td>No demographics</td> <td></td> <td></td> <td><b>4</b></td> <td><b>30.1</b></td> </tr> <tr> <td><input type="checkbox"/> 8.</td> <td>Not within expected values</td> <td></td> <td></td> <td><b>4</b></td> <td><b>30.1</b></td> </tr> <tr> <td>Unit</td> <td><b>mkmol/L</b></td> <td>Decimal Places</td> <td><b>1</b></td> <td></td> <td></td> </tr> </table>						Value/Flag:	<input type="text"/> $\nabla$	Level L:	<input type="text"/>	Level H:	<input type="text"/>	Specific Ranges:							Sex	Year	Month	Year	Month	<input type="checkbox"/> 1.	<input type="text"/> $\nabla$	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="checkbox"/> 2.	<input type="text"/> $\nabla$	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="checkbox"/> 3.	<input type="text"/> $\nabla$	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="checkbox"/> 4.	<input type="text"/> $\nabla$	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="checkbox"/> 5.	<input type="text"/> $\nabla$	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="checkbox"/> 6.	<input type="text"/> $\nabla$	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="checkbox"/> 7.	No demographics			<b>4</b>	<b>30.1</b>	<input type="checkbox"/> 8.	Not within expected values			<b>4</b>	<b>30.1</b>	Unit	<b>mkmol/L</b>	Decimal Places	<b>1</b>																																												
Value/Flag:	<input type="text"/> $\nabla$	Level L:	<input type="text"/>	Level H:	<input type="text"/>																																																																																																																		
Specific Ranges:																																																																																																																							
	Sex	Year	Month	Year	Month																																																																																																																		
<input type="checkbox"/> 1.	<input type="text"/> $\nabla$	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="text"/>																																																																																																																		
<input type="checkbox"/> 2.	<input type="text"/> $\nabla$	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="text"/>																																																																																																																		
<input type="checkbox"/> 3.	<input type="text"/> $\nabla$	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="text"/>																																																																																																																		
<input type="checkbox"/> 4.	<input type="text"/> $\nabla$	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="text"/>																																																																																																																		
<input type="checkbox"/> 5.	<input type="text"/> $\nabla$	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="text"/>																																																																																																																		
<input type="checkbox"/> 6.	<input type="text"/> $\nabla$	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="text"/>																																																																																																																		
<input type="checkbox"/> 7.	No demographics			<b>4</b>	<b>30.1</b>																																																																																																																		
<input type="checkbox"/> 8.	Not within expected values			<b>4</b>	<b>30.1</b>																																																																																																																		
Unit	<b>mkmol/L</b>	Decimal Places	<b>1</b>																																																																																																																				
<table border="0"> <tr> <td colspan="2">Parameters</td> <td colspan="4">Calibration Parameters</td> </tr> <tr> <td>Calibrators</td> <td>Calibration Specific</td> <td colspan="4">STAT Table Calibration</td> </tr> <tr> <td>General</td> <td>ISE</td> <td colspan="4"></td> </tr> </table>						Parameters		Calibration Parameters				Calibrators	Calibration Specific	STAT Table Calibration				General	ISE																																																																																																				
Parameters		Calibration Parameters																																																																																																																					
Calibrators	Calibration Specific	STAT Table Calibration																																																																																																																					
General	ISE																																																																																																																						
Test Name: <b>Fe</b> < > Type: <b>Serum</b> <input type="radio"/> Use Serum Cal.																																																																																																																							
<table border="0"> <tr> <td colspan="2">Calibration Type: <b>AB</b> <math>\nabla</math></td> <td colspan="2">Formula: <b>Y=AX+B</b> <math>\nabla</math></td> <td>Counts: <b>2</b> <math>\nabla</math></td> <td></td> </tr> <tr> <td colspan="6">&lt;Calibrator Parameters&gt;</td> </tr> <tr> <td>Calibrator</td> <td>OD</td> <td>Conc</td> <td>Low</td> <td>High</td> <td>Slope Check</td> </tr> <tr> <td>Point 1:</td> <td><b>TruCal U</b></td> <td><b>*</b></td> <td><b>-99999</b></td> <td><b>99999</b></td> <td><b>None</b> <math>\nabla</math></td> </tr> <tr> <td>Point 2:</td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Point 3:</td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Point 4:</td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Point 5:</td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Point 6:</td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Point 7:</td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Point 8:</td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Point 9:</td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Point 10:</td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td colspan="2">&lt;Point Cal. For</td> <td>No. of Correction Points</td> <td><input type="text"/> <math>\nabla</math></td> <td>Use Master Curve</td> <td><input type="text"/> <math>\nabla</math> <input type="radio"/> Lot Calibration</td> </tr> <tr> <td colspan="6">Master Curve&gt;</td> </tr> <tr> <td>Calibrator</td> <td>OD</td> <td>Conc</td> <td>Low</td> <td>High</td> <td>Stability</td> </tr> <tr> <td>Point 1:</td> <td></td> <td></td> <td></td> <td></td> <td>Reagent Blank <input type="text"/> Day <input type="text"/> Hour</td> </tr> <tr> <td>Point 2:</td> <td></td> <td></td> <td></td> <td></td> <td>Calibration <input type="text"/> Day <input type="text"/> Hour</td> </tr> <tr> <td colspan="2">MB Type Factor:</td> <td><input type="text"/></td> <td>1-Point Calibration Point</td> <td><input type="text"/> <math>\nabla</math></td> <td><input type="radio"/> with Conc-0</td> </tr> </table>						Calibration Type: <b>AB</b> $\nabla$		Formula: <b>Y=AX+B</b> $\nabla$		Counts: <b>2</b> $\nabla$		<Calibrator Parameters>						Calibrator	OD	Conc	Low	High	Slope Check	Point 1:	<b>TruCal U</b>	<b>*</b>	<b>-99999</b>	<b>99999</b>	<b>None</b> $\nabla$	Point 2:						Point 3:						Point 4:						Point 5:						Point 6:						Point 7:						Point 8:						Point 9:						Point 10:						<Point Cal. For		No. of Correction Points	<input type="text"/> $\nabla$	Use Master Curve	<input type="text"/> $\nabla$ <input type="radio"/> Lot Calibration	Master Curve>						Calibrator	OD	Conc	Low	High	Stability	Point 1:					Reagent Blank <input type="text"/> Day <input type="text"/> Hour	Point 2:					Calibration <input type="text"/> Day <input type="text"/> Hour	MB Type Factor:		<input type="text"/>	1-Point Calibration Point	<input type="text"/> $\nabla$	<input type="radio"/> with Conc-0
Calibration Type: <b>AB</b> $\nabla$		Formula: <b>Y=AX+B</b> $\nabla$		Counts: <b>2</b> $\nabla$																																																																																																																			
<Calibrator Parameters>																																																																																																																							
Calibrator	OD	Conc	Low	High	Slope Check																																																																																																																		
Point 1:	<b>TruCal U</b>	<b>*</b>	<b>-99999</b>	<b>99999</b>	<b>None</b> $\nabla$																																																																																																																		
Point 2:																																																																																																																							
Point 3:																																																																																																																							
Point 4:																																																																																																																							
Point 5:																																																																																																																							
Point 6:																																																																																																																							
Point 7:																																																																																																																							
Point 8:																																																																																																																							
Point 9:																																																																																																																							
Point 10:																																																																																																																							
<Point Cal. For		No. of Correction Points	<input type="text"/> $\nabla$	Use Master Curve	<input type="text"/> $\nabla$ <input type="radio"/> Lot Calibration																																																																																																																		
Master Curve>																																																																																																																							
Calibrator	OD	Conc	Low	High	Stability																																																																																																																		
Point 1:					Reagent Blank <input type="text"/> Day <input type="text"/> Hour																																																																																																																		
Point 2:					Calibration <input type="text"/> Day <input type="text"/> Hour																																																																																																																		
MB Type Factor:		<input type="text"/>	1-Point Calibration Point	<input type="text"/> $\nabla$	<input type="radio"/> with Conc-0																																																																																																																		

Диапазон нормальных значений указан в соответствии с рекомендациями производителя реагентов к набору и может быть изменен пользователем в соответствии с местными требованиями.

Калибровка линейная, АВ (калибратор TruCal U, или стандарт из набора).

\*-вводится из паспорта к калибратору (стандарт)

Контроль TruLab N и TruLab P.